

---

Ingenuity in  
infection control  
since 1830

A large, dark blue rectangular area with a complex, abstract pattern of concentric, wavy lines in a lighter shade of blue, creating a sense of depth and movement. The lines are more densely packed in the center and become more sparse towards the edges.

# Autoclave Accessories

---

## User Manual

---

## ACCESSORIES COVERED BY THIS INSTRUCTION

8704019: Stainless Steel Tray (45cm x 18cm)










8704066: Pouch Rack

8727121: Tray Lifter

## SAFETY WARNINGS

- Refer to the autoclave instructions for use for the full list of safety warnings applicable to the product for which these accessories are intended.
- To prevent injury risk due to dropping the load, only use the Eschmann tray lifter to load and unload the Eschmann pouch rack and Eschmann trays.
- To prevent injury risk due to dropping the load, do not use the Eschmann tray lifter to load or unload non Eschmann accessories.
- To prevent burns and scalds when loading and unloading the autoclave, the user should avoid contact with hot accessible surfaces of the accessories.
- To prevent burns and scalds, avoid contact with the hot internal surfaces of the chamber and door.
- To prevent scalds, avoid contact with steam and hot water that may exit the chamber when opening the door after a cycle.
- Items to be sterilised must be cleaned prior to loading into the autoclave. If items to be sterilised are not clean, this may compromise the sterilisation process.
- Items to be sterilised must be suitable and correctly loaded into the autoclave, in accordance with the autoclave Instructions for Use. Incorrect loading or overloading may compromise the sterilisation process.

## PRODUCT INFORMATION AND SAFETY SYMBOLS

Product Information and safety Symbols	
 or 	Attention: See the instructions for use
	Consult the instructions for use
	Date of manufacture
	Medical Device
	Lot number
	EC authorised representative
	The CE marking of this product certifies that it complies with EU Medical Device Regulations 2017/745
	The UKCA marking of the product certifies that it complies with the Medical Devices Regulations 2002 (SI 618), as amended by the EU Exit Regulations 2019 (SI 791) and 2020 (SI 1478)

# INSTRUCTIONS

---

- Remove all packaging materials from the accessory prior to use.
- Refer to the autoclave instructions for use for the full operating instructions applicable to the product and use of accessories.
- The Eschmann tray lifter must be used to load and unload Eschmann trays and the Eschmann pouch rack.


## Using the Eschmann Tray lifter with Eschmann trays and HFiT Cassettes:

To load Eschmann trays and HFiT cassettes, position the half-round base UNDER the item to be lifted. Hook onto the item using one of the pairs of top clips.


Use the upper pair of top clips for HFiT - hold at the side or end of the cassette.

Use the lower pair of top clips for trays.

### HFiT Cassettes

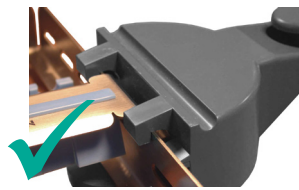


HFiT cassette gripped on its side.

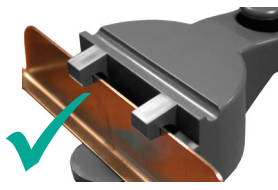


HFiT cassette gripped on its end.

### Trays

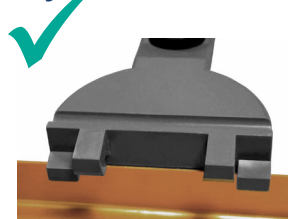

Tray in bottom clips  
CORRECT

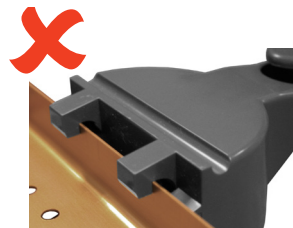


Tray in top clips  
INCORRECT

## Using the Eschmann Tray lifter with the Eschmann Pouch Rack or Porous Load Basket:

### Pouch Rack/ Porous Load Basket

Locate the end bar of the pouch rack or basket into the tray lifter's top clip.

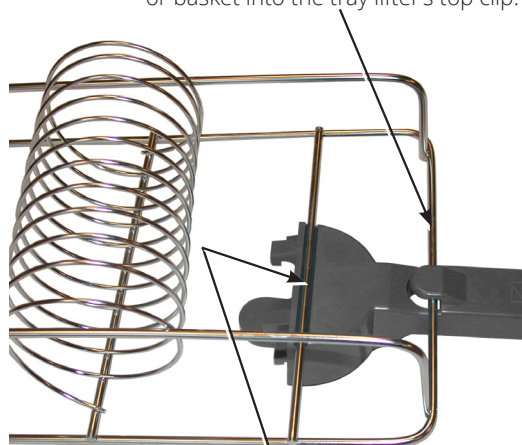

Locate this bar into the channel on the tray lifter

## Service

Eschmann products must only be serviced by Eschmann, or Eschmann trained engineers. We recommend a program of ongoing service and maintenance using only Eschmann spare parts.

Eschmann products are supported by a worldwide network of fully trained engineers, offering high quality Eschmann spare parts. For further information on the range of Service Contracts available, please contact your local Eschmann representative.

Eschmann can be contacted during normal office hours. Please use the contact details below.

## Safety Reporting

Any serious incident that occurs with this medical device should be reported to Eschmann Technologies Limited and your local Competent Authority

## Warranty

The product is warranted for a minimum period of 12 months to be free from defects in materials and workmanship at the time of delivery.

Eschmann will be under no liability for any defect arising from fair wear and tear, negligence, wilful damage, misuse, abnormal working conditions, failure to follow the manufacturer's instructions, unauthorised alteration or repair, lost profits, commercial loss, economic loss, or loss arising from personal injury. We may, at our discretion, raise a charge for any faults repaired that fall outside the warranty cover. Where charges are necessary, replacement parts will be charged at manufacturers' list prices and labour will be charged at the prevailing hourly rate. Repairs performed by Eschmann carry a 3-month parts and labour warranty.

The details in this manual are correct at time of going to press and supersede details previously published elsewhere. Information, technical specifications and statements made in this publication may be subject to change without prior notice

## EC Authorised Representative

Casus Europe B.V. Lange Viestraat 2b, 3511 BK Utrecht, The Netherlands

---

### Eschmann Technologies Limited

Eschmann House, 15 Peter Road, Lancing,  
West Sussex, BN15 8TJ United Kingdom

**t:** +44 (0)1903 753322

**e:** [info@eschmann.co.uk](mailto:info@eschmann.co.uk)

**w:** [eschmann.co.uk](http://eschmann.co.uk)

Ingenuity in  
infection control  
since 1830

